


Additional Instructions

IMPORTANT! The Neuro jacks are all round in shape so please be sure to plug the Charger Plug in the correct socket. Otherwise, it may cause damage to the Controller.

Here are additional instructions for the Neuro regarding proper charging:

- The Neuro is dual voltage (220 V or 110V) and has a North American plug. Outside of North America, use a travel adaptor for charging.
- To charge the controller, **the Charger Plug must be inserted into the socket as indicated below (See #1)**, as shown in the diagram below. Inserting the larger headset or applicator plugs into this socket will cause damage to components near this location resulting in problems with charging.
- Please charge the controller for another 6 hours before use.

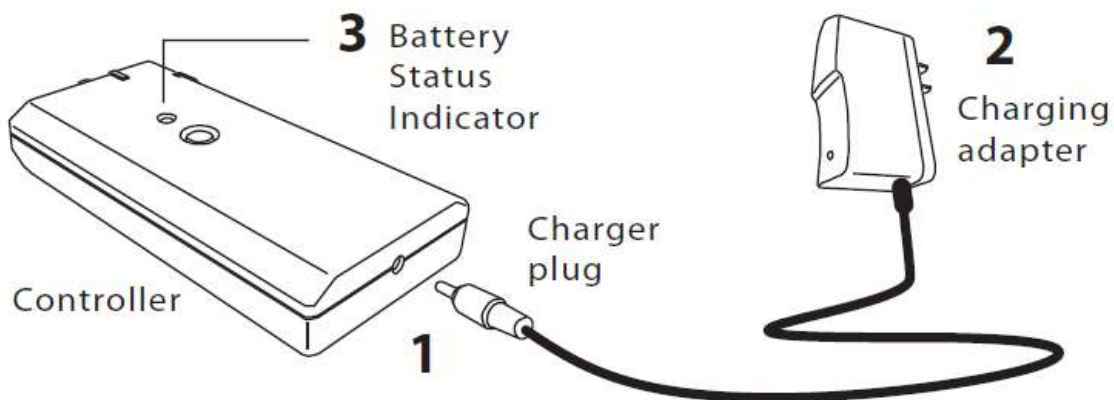

- A fully charged controller is sufficient for 16 cycles of use.

Information on Vielight Nasal Applicators

As a precaution against damaging the applicator, remove the applicator from the nose by holding onto the clip, not by pulling at the cable.

To sanitize the nasal applicator, please wipe gently with an alcohol or hydrogen peroxide wipe and use a UV sanitizer.

Submerging the nasal applicator in liquid will damage it. Do not cover the applicator with any other material during use.

Vielight

NeURO USER'S GUIDE

Additional Instructions

The Vielight Neuro recommended usage is once every two or three days.
The treatment time is 20 minutes.



Click on image below to be directed to instructional video.

